
nuance | Voice Selector hearing | Study


Easy to listen Easy to focus

Quick start guide

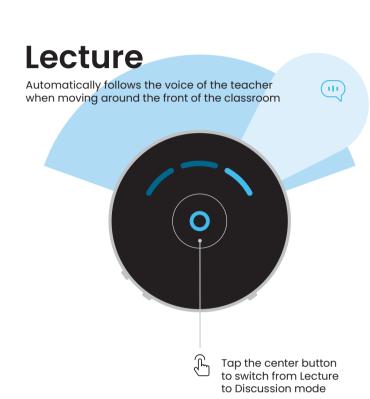

Discussion

Automatically follows the voice of the current dominant speaker in every direction

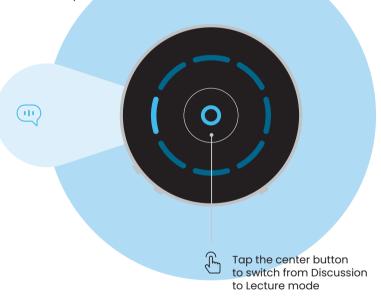


Conversation

Manually select the direction of up to 2 speakers




Tap the segment in the direction of a particular speaker

Battery and charging

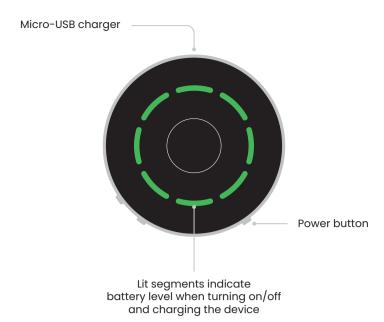

Volume


For optimal performance, set volume level so voices sound natural. High volumes may amplify undesired voices and background noise

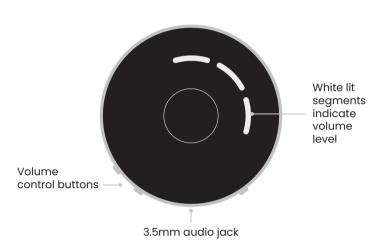

Warning: with continued use at a high volume, your ears can become accustomed to the sound level, which may result in permanent damage to your hearing without any noticeable discomfort.

Headphones

You can use any headphones with 3.5mm audio plug

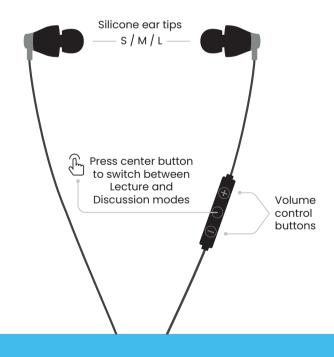

nuance | Voice Selector
hearing | Study

Good luck with your studies!

We're here to help you with any question or issue

support@nuancehear.com +972 77 901 8642